



Easy as 1, 2, 3 when doing an ECG with the NK 1250A or 1350A

1. Turn power on 2. Enter Patient Info 3. Push Start button

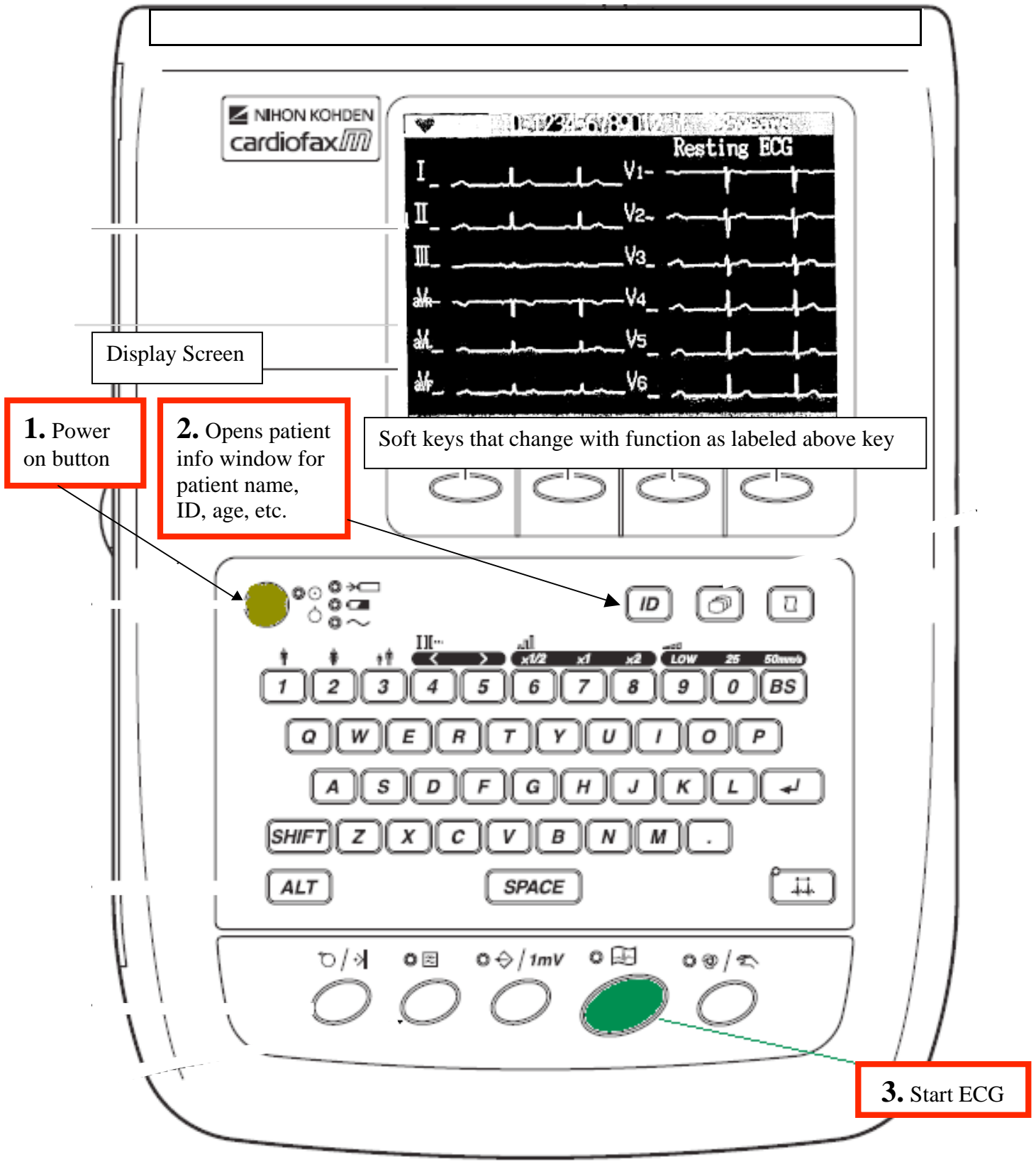

Basic Operations of Nihon Kohden 1250A and 1350A ECG machines

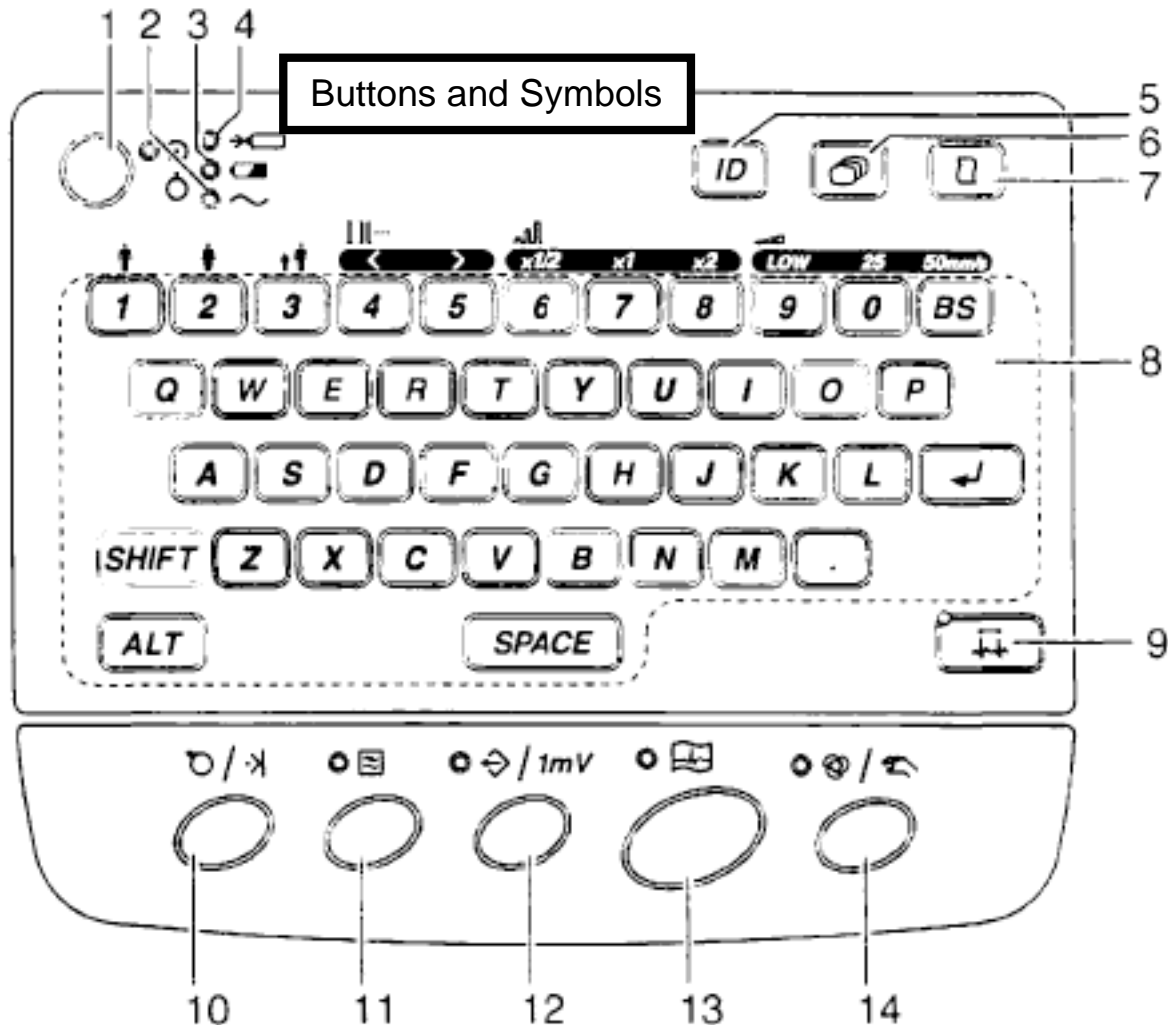

1. Power switch
2. Lights up when connected to AC power source
3. Lights up when operating on battery as power source and indicates charge level
4. Lights up when battery is charging
5. ID button – opens Patient Information Screen
6. Opens Main Menu Screen
7. Review key
8. Keyboard
9. Rhythm key
10. Feed / Mark key
11. Filter key Lamp on = EMG filter on
12. Copy button
13. Start / Stop button
14. Auto / Manual button lamp on = Auto

Unlikely needed resource list:

Technical Support at NK 800-325-0283

Local Support:
 Generic Medical Solutions
 Dan Walton
 Office: 847-397-6111
 Cell: 847-828-9800